

## Windows 8 Driver Installation

Ensure the GENEActiv driver files are easily accessible (e.g. CD or Desktop) and that a cradle is connected containing one device.

1. Select the settings cog wheel and then 'Change PC settings' at the bottom.
2. Select the 'General' tab and then click the 'Restart now' tab at the bottom of the list under 'Advanced start-up'.
3. The machine will pause then give 3 options, choose 'Troubleshoot'.
4. Select 'Advanced options' and then 'Start-up Settings'.
5. We need to 'Disable driver signature enforcement' so click 'Restart' to proceed.
6. On restart, press F7 to select option 7.

Once the PC has fully restarted:

7. Select: Control Panel -> Hardware and Sound -> Devices and Printers
8. Then right-click the GENEActiv\_01s
9. Select: Properties Menu -> Hardware Tab -> Properties Button -> Change Settings Button -> Driver Tab -> Update Driver Button -> Browse
10. Then select the GENEActiv driver files and click OK.
11. If a security warning pop-up appears then click continue.

Once this is complete, start the GENEActiv PC software and the device should be recognised. All future device additions will use this driver and the process does not need to be repeated.